

CUTIMED® SORBACT® HYDROACTIVE B

FOR REMOVAL OF NECROTIC TISSUE AND INFECTION MANAGEMENT

- Hydropolymer gel matrix helps maintain a moist wound environment
- Stimulates autolytic debridement of fibrin and necrotic tissue
- Helps prevent maceration
- Ideal for absorption of thin exudate



FOUR ADVANTAGES IN ONE DRESSING

ABSORB



HYDRATE



DEBRIDE




SORBACT®

4-IN-1: ABSORB. HYDRATE. DEBRIDE. + CUTIMED® SORBACT®

This innovative hydropolymer gel dressing provides the performance of absorbency with balanced hydration. Cutimed® Sorbact® binds and removes bacteria, versus killing like antimicrobial dressings and allows for natural wound healing. Combined, these features promote autolytic debridement, help reduce the bacterial load, and offer fast, effective wound healing.

PROOF OF WOUND HEALING

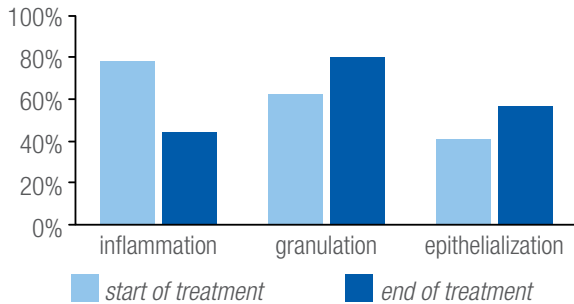

Data of clinical observations (n = 100) indicated a noticeable change of phases of wound healing. Inflammatory phase is decreased and the proliferative phase is increased (granulation and epithelialization) after 10 days of Cutimed® Sorbact® Hydroactive treatment.¹

HYDRATION OF PERI-WOUND SKIN

Water and glycerol allow the dressing to hydrate the wound and peri-wound area.




Improvement in peri-wound skin within one week.

IT'S ALL ABOUT BALANCE

Think of the dressing as a cycle of absorbing exudate and stimulating perfusion. Through osmosis, fluid from the wound is pulled into the absorbent polymers. Therefore, this stimulates the wound and draws fluid from surrounding tissues aiding the body's natural process to clear debris and necrotic tissue from the wound.

**Absorbent
Polymer**


Cutimed® Sorbact® Hydroactive B Dressing*	Size	Wound Pad Size	REF No.	Dressings/ Box
	2.8 × 3.3 in. (7 × 8.5 cm)	2 × 2.6 in. (5 × 6.5 cm)	7993300	10
	5.5 × 5.5 in. (14 × 14 cm)	3.9 × 3.9 in. (10 × 10 cm)	7993302	10
	7.5 × 7.5 in. (19 × 19 cm)	5.9 × 5.9 in. (15 × 15 cm)	7993303	10
	5.5 × 9.4 in. (14 × 24 cm)	3.9 × 7.9 in. (10 × 20 cm)	7993304	10

1 Hallern von, B. (2012, May). Absorbent DACC coated dressings, a new approach in treating wound infection: A multi-centre study. Poster presented at European Wound Management Association (EWMA) Conference, Vienna, Austria.

Sorbact® is a registered trademark of ABIGO Medical AB.
Please refer to the product label and/or package insert for full instructions on safe use of these products.



Cutimed®,
an Essity brand

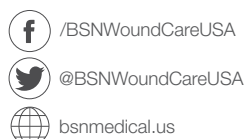

BSN Medical Inc., an Essity company
5825 Carnegie Blvd., Charlotte, NC 28209-4633
Tel. (+1) 704 554 9933 Fax (+1) 800 835 4325
To order toll-free: Tel. (+1) 800 537 1063

62061 R4 ©2020 BSN Medical Inc. REV G20